

## Urine Collection



### Grey Top Urine Culture Transport Kit

**Tests:** Urine Culture

**Test code:** UC

**Stability:** Unpreserved urine specimens are stable at room temperature for up to 1 hour, or if refrigerated, up to 24 hours. An unpreserved urine can be placed in a grey top urine culture transport tube within 24 hours to increase stability to 72 hours.

**Collection information:** Clean Catch, Mid stream

**Transport:** Refrigerated



### Yellow Top Urine Transport Kit

**Tests:** URINALYSIS, ROUTINE WITH REFLEX TO MICRO, URINALYSIS W/RFLX TO MICRO&/OR CULTURE, URINE MICROSCOPY ONLY

**Test Code:** US, USM, USR

**Stability:** Urine specimens are stable at room temperature for up to 1 hour, or if refrigerated, up to 24 hours.

**Collection information:** Clean Catch, Mid stream collection

**Transport:** Refrigerated